Instructions for use and safety recommendations

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Packung zu 1, 3, 6, 12

Intensiv Ortho-Strips One-Sided, Ref. OS15L, links
Intensiv Ortho-Strips One-Sided, Ref. OS15R, rechts
Intensiv Ortho-Strips One-Sided, Ref. OS25L, links
Intensiv Ortho-Strips One-Sided, Ref. OS25R, rechts
Intensiv Ortho-Strips One-Sided, Ref. OS40L, links
Intensiv Ortho-Strips One-Sided, Ref. OS40R, rechts
Intensiv Ortho-Strips One-Sided, Ref. OS60L, links
Intensiv Ortho-Strips One-Sided, Ref. OS60R, rechts
Intensiv Ortho-Strips One-Sided Set, Ref. 064
(OS15L, OS15R, OS25L, OS25R, OS40L, OS40R)

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Packung zu 1, 3, 6, 12

Intensiv Ortho-Strips, Ref. OS90
Intensiv Ortho-Strips, Ref. OS60
Intensiv Ortho-Strips, Ref. OS40
Intensiv Ortho-Strips, Ref. OS25
Intensiv Ortho-Strips, Ref. OS15
Intensiv Ortho-Strips Set, Ref. 060A (OS90, OS40, OS25, OS15)
Intensiv Ortho-Strips Set, Ref. 060B (OS60, OS40, OS25, OS15)

Packung zu 3, 6

Intensiv Ortho-Strips Coarse, Double-Sided, Ref. OS80C
Intensiv Ortho-Strips Coarse, One-Sided, Ref. OS80C-R, rechts
Intensiv Ortho-Strips Coarse, One-Sided, Ref. OS80C-L, links

Packung zu 3, 6

Intensiv Ortho-Strips Opener, Double-Sided, Ref. OS08OP
Intensiv Ortho-Strips Opener, One-Sided, Ref. OS08OP-R, rechts
Intensiv Ortho-Strips Opener, One-Sided, Ref. OS08OP-L, links

GB

Intensiv Ortho-Strips One-Sided

University of Zurich

Oscillating instruments for the monolateral interdental reduction of the tooth size in orthodontics.

To be used in combination with the contra-angles:

- Intensiv Swingle, WG-69 LT Ortho, with light
- Intensiv Swingle, WG-69 A Ortho, without light

Alternatively:

- KaVo head 61 LG
- KaVo contra-angle E61C

Product description

- One sided diamond-coated oscillating metal strip
- Strip height: 3.7 mm
- Length of the diamond-coated part: 13 mm
- Sterilizable and reusable

Indications

- Opening of the interdental space monolateral
- Contouring and polishing of the enamel

Recommendations for use

- Apply Intensiv Ortho-Strips One-Sided in interproximal areas according to the IPR enamel reduction protocol for orthodontic treatments.
- A pre-opening of tight contact points using Intensiv Ortho-Strips Opener is suggested, Ref. OS08OP-R/6, right or Ref. OS08OP-L/6, left.
- In case of very tight contact points separate the teeth by placing a wooden wedge cervically. This makes stripping more comfortable both for operator and patient.
- Set the micro motor at 40'000 RPM/min max. With Intensiv Swingle this corresponds to 20'000 oscillations per minute approx. (2:1).
- Activate water spray during the entire treatment (minimum 50 ml/min).
- Regulate water flow in case of water excess or shortage.
- Insert the strip in a cervical direction while the contra-angle is moving. This prevents the instruments from jamming.
- Apply a load not greater than 100 grams, up to a maximum bending of the strip of approximately 1 mm.
- Do not stop the strip in the interdental space.
- Remove the strip from the interdental space whilst the contra-angle is moving.

Maintenance and Sterilization

- Clean the instruments and remove debris after each use, so as to maintain their abrasive properties.
- Instruments are delivered non sterile. They must be sterilized prior to first use and disinfected, cleaned, and sterilized after each use.
- Use a single-pulsed or fractionated vacuum autoclave and subvacuum drying. Chemiclave sterilizers may also be used. Hot air sterilizers are not suitable for diamond instruments.
- Sterilization must be carried out according to validated procedures.
- Concerning the sterilization process we refer to the ISO standard 17664.
We suggest to follow the indications below:
 - Cycles at 134°C:
Tmin = 134°C – Tmax = 138°C
Pressure = 3.15 bar abs
Time = 4 min (raisable)
 - Cycles at 121°C:
Tmin = 121°C – Tmax = 125°C
Pressure = 2.10 bar abs
Time = 16 min (raisable)

Risk Warnings

- Avoid bending and levering: use of excessive force (strong levering effect with the contra-angle) could lead to the breakage of the instrument.
- Avoid jamming or levering actions when oscillating, as this increases the risk of instrument breakage.
- Oscillating over 20'000 oscillations per minute could cause breakage of the instrument.
- Avoid temperatures above 180°C which may affect the durability of the instrument.
- Avoid applying loads greater than 100 grams as this could cause damage to the instrument and/or the tooth.
- To ensure traceability of the instruments during their entire application, we recommend keeping the packaging.

Packaging of 1, 3, 6, 12

Intensiv Ortho-Strips One-Sided, Ref. OS15L, left
Intensiv Ortho-Strips One-Sided, Ref. OS15R, right
Intensiv Ortho-Strips One-Sided, Ref. OS25L, left
Intensiv Ortho-Strips One-Sided, Ref. OS25R, right
Intensiv Ortho-Strips One-Sided, Ref. OS40L, left
Intensiv Ortho-Strips One-Sided, Ref. OS40R, right
Intensiv Ortho-Strips One-Sided, Ref. OS60L, left
Intensiv Ortho-Strips One-Sided, Ref. OS60R, right
Intensiv Ortho-Strips One-Sided Set, Ref. 064
(OS15L, OS15R, OS25L, OS25R, OS40L, OS40R)

Additional models

Packaging of 1, 3, 6, 12

Intensiv Ortho-Strips, Ref. OS90
Intensiv Ortho-Strips, Ref. OS60
Intensiv Ortho-Strips, Ref. OS40
Intensiv Ortho-Strips, Ref. OS25
Intensiv Ortho-Strips, Ref. OS15
Intensiv Ortho-Strips Set, Ref. 060A (OS90, OS40, OS25, OS15)
Intensiv Ortho-Strips Set, Ref. 060B (OS60, OS40, OS25, OS15)

Packaging of 3, 6

Intensiv Ortho-Strips Coarse, Double-Sided, Ref. OS80C
Intensiv Ortho-Strips Coarse, One-Sided, Ref. OS80C-R, right
Intensiv Ortho-Strips Coarse, One-Sided, Ref. OS80C-L, left

Packaging of 3, 6

Intensiv Ortho-Strips Opener, Double-Sided, Ref. OS08OP
Intensiv Ortho-Strips Opener, One-Sided, Ref. OS08OP-R, right
Intensiv Ortho-Strips Opener, One-Sided, Ref. OS08OP-L, left

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Intensiv Ortho-Strips One-Sided, Ref. OS15L, sinistra
Intensiv Ortho-Strips One-Sided, Ref. OS15R, destra
Intensiv Ortho-Strips One-Sided, Ref. OS25L, sinistra
Intensiv Ortho-Strips One-Sided, Ref. OS25R, destra
Intensiv Ortho-Strips One-Sided, Ref. OS40L, sinistra
Intensiv Ortho-Strips One-Sided, Ref. OS40R, destra
Intensiv Ortho-Strips One-Sided, Ref. OS60L, sinistra
Intensiv Ortho-Strips One-Sided, Ref. OS60R, destra
Intensiv Ortho-Strips One-Sided Set, Ref. 064
(OS15L, OS15R, OS25L, OS25R, OS40L, OS40R)

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Intensiv Ortho-Strips Coarse, Double-Sided, Ref. OS80C
Intensiv Ortho-Strips Coarse, One-Sided, Ref. OS80C-R, destra
Intensiv Ortho-Strips Coarse, One-Sided, Ref. OS80C-L, sinistra

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Intensiv Ortho-Strips Opener, Double-Sided, Ref. OS08OP
Intensiv Ortho-Strips Opener, One-Sided, Ref. OS08OP-R, destra
Intensiv Ortho-Strips Opener, One-Sided, Ref. OS08OP-L, sinistra

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Intensiv Ortho-Strips One-Sided, Ref. OS15L, izquierda
Intensiv Ortho-Strips One-Sided, Ref. OS15R, derecha
Intensiv Ortho-Strips One-Sided, Ref. OS25L, izquierda
Intensiv Ortho-Strips One-Sided, Ref. OS25R, derecha
Intensiv Ortho-Strips One-Sided, Ref. OS40L, izquierda
Intensiv Ortho-Strips One-Sided, Ref. OS40R, derecha
Intensiv Ortho-Strips One-Sided, Ref. OS60L, izquierda
Intensiv Ortho-Strips One-Sided, Ref. OS60R, derecha
Intensiv Ortho-Strips One-Sided Set, Ref. 064
(OS15L, OS15R, OS25L, OS25R, OS40L, OS40R)

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Intensiv Ortho-Strips Coarse, Double-Sided, Ref. OS80C
Intensiv Ortho-Strips Coarse, One-Sided, Ref. OS80C-R, derecha
Intensiv Ortho-Strips Coarse, One-Sided, Ref. OS80C-L, izquierda

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Intensiv Ortho-Strips Opener, Double-Sided, Ref. OS08OP
Intensiv Ortho-Strips Opener, One-Sided, Ref. OS08OP-R, derecha
Intensiv Ortho-Strips Opener, One-Sided, Ref. OS08OP-L, izquierda